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**EN** • Instructions of use: Protective glove for single use, waterproof and airproof, designed for use without mechanical, electrical or thermal hazards. This product provides resistance to certain chemicals (see table). This product protects from bacteria, mold and virus. Compatible with all types of food for a short time. Do not use out of the scope of use. Do not use with corrosive, toxic or irritant chemical products other than those mentioned in the performances without prior tests. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. Performance levels are based on the results of laboratory tests, which do not necessarily reflect real conditions in the workplace. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. This information does not reflect the actual duration of protection at the workplace or the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only and relates only to the chemical tested. It can be different if the chemical is used in a mixture. When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals degradation can be the most important factor to consider in selection of chemical resistant gloves. These gloves do not contain any substances known to be carcinogenic or toxic. LATEX: Parts which may come into contact with wearer's skin and could cause allergic reactions to susceptible individuals. In this case, stop all uses and seek medical advice. Ensure your gloves are intact before and during using it and replace if necessary. Store in a cool, dry place away from frost and light in their original packaging. Transport instructions: To be transported in its original packaging. Storage life: We recommend 3 years of storage from production date. Meets the essential requirements of REGULATION (EU) 2016/425 and standards EN420:2003+A1:2009 (Dexterity 5), EN ISO 374-1:2016 (Type C), EN ISO 374-5:2016 and EN455-1/2/3. Tested to EN374-2:2014 (airtight and watertight), EN16523-1:2015 (Permeation resistance: K = 40% caustic soda, T = 37% formaldehyde), EN374-4:2013 (Resistance to degradation) and ISO 16604 (VIRUS). The declaration of conformity can be found on the website [www.deltaplus.eu](http://www.deltaplus.eu) in the data of the product.

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